

Leaflet for your measurements of intraocular pressure with the eyemate-IO system

Patient-ID: _____

Daily home measurements

- Measurements with eyemate-IO should be performed at least **4 times** a day:
 1. **Morning** between 7 and 9 a.m.
 2. **Noon** between 11:30 a.m. and 1:30 p.m.
 3. **Afternoon** between 3 and 5 p.m.
 4. **Evening** between 7 and 11 p.m.

IOP daily routine:

Please note down your activity during the day, especially your drug intake if being performed at the same time as the 4 main measurements.

You may perform additional measurements as soon as you wish. If the measurement has been taken due to special events/activities, it will be very helpful to give us a short notice about the reason.

Additional measurements should be taken one hour before and one after the application of your glaucoma eye drops.

Documenting all your activities as fully as possible in your patient diary will help you, your doctor and the scientific community to better understand the disease of glaucoma. Thank you in advance for your help!

Your eye doctor has specified a maximum intraocular pressure for you that should not be exceeded more than __ times a day.¹

Your maximum IOP: _____

Please contact your eye doctor if your intraocular pressure exceeds the maximum pressure more than ____ days!

Your eye doctor: _____

Phone: _____

¹ Applicable for measurements in rest and sitting!